

Notes

Looking after yourself

Your Doctor or Nurse should be told if you notice any change in the smell, colour or clarity of your urine or if blood or particles are seen.

Delivery

Catheters are supplied individually packaged in a sterile pouch. The sterility of the products is only guaranteed as long as the packaging unit is unopened and undamaged, and no longer than indicated on the label.

Storage Instruction

The products shall be stored in their original packaging. Keep away from sunlight and keep dry. Do not use if the product sterilisation barrier or its packaging is compromised.

Warnings:

The products are for single use only.



Single use: Do not reuse, reprocess or re-sterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death. Reprocessing of medical devices intended for single use only may result in degraded performance or a loss of functionality.



REF 5143 and 5105

These catheters contain natural rubber latex (see product label), which may cause allergic reactions. Under no circumstances should these products be used in patients with a latex allergy.

Sterilization method: see product label

STERILE/EO

STERILE R

Explanation of important symbols and markings on the product label.

REORDER	Complete product order number, consisting of the order number "REF" and the product size "SIZE"
REF	Catalogue number
SIZE	Product size
Ch.	1 Ch. = 1/3mm
O.D.	Outer diameter
Length	Catheter length
Qty	Quantity, Number of items
LOT	Batch code
	Date of manufacture
	Use by
STERILE/EO	Sterilized using ethylene oxide
STERILE R	Sterilized using irradiation
	Manufacturer
EC REP	EU Authorized Representative
	Caution, Attention, Important, Note
	Consult instructions for use
	Do not reuse
	Do not resterilize
	Not made with natural rubber latex
	This product contains natural rubber latex
	Do not use if package is damaged
	Keep dry
	Keep away from sunlight

For further information please contact us on:

TELEFLEX MEDICAL EUROPE Ltd.


IDA Business & Technology Park
Dublin Road, Athlone, Co. Westmeath, Ireland
Phone +353 (0) 906 460 800
Fax +353 (0) 143 707 73

www.teleflex.com

Teleflex®

Teleflex Medical Sdn. Bhd.
Lot PT 2577, Jalan Perusahan 4,
34600 Kamunting, Perak, Malaysia
Made in Malaysia

Teleflex Medical
IDA Business & Technology Park,
Dublin Road, Athlone,
Co. Westmeath, Ireland

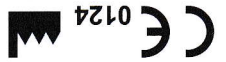

EC REP