

Riplex Jaques Nelaton Intermittent Catheters

Warning: Read all package insert warnings, precautions, and instructions prior to use. Failure to do so may result in severe patient injury or death.

Indication:

Intermittent Catheterisation (e.g. neurogenic bladder dysfunction)

Contraindication:

- Insurmountable urethral obstruction
- Urethral injuries
- Urethral inflammation
- Latex allergy (for products containing natural rubber latex) in this instance only 5143 and 5105 contain latex

Direction for Use

• Male

Before each catheterisation make sure you have all you need together e.g. a new catheter, a lubricating gel (if used), cleansing materials.

1. Wash your hands thoroughly and cleanse the genital area (as you have been shown by your Health Care Professional).
2. Position yourself as you have been shown.
3. Remove the catheter from the packaging. You may wish to either insert lubricant into your urethra or apply the lubricant to the catheter, prior to insertion.
4. Pull back the foreskin, then holding your penis up, with your other hand push the catheter gently into the urethra.
5. Insert the catheter until urine begins to flow. You may feel more resistance as the catheter enters the bladder. Do not worry, relax and push gently. Once urine flows, maintain the position of the catheter and try to avoid further insertion.
6. When the urine stops flowing, gently push the catheter a little further into the bladder and wait to see if more urine flows.
7. When you are happy that your bladder is empty, slowly withdraw the catheter. If more urine flows, stop and wait for it to cease, and then continue to withdraw.
8. Wash your hands thoroughly and discard the catheter in line with your local waste recommendations. Most Councils recommend first rinsing the used catheter and either seal in a plastic bag or wrap in newspaper before putting in the normal household waste.
 - Never burn any used catheters on domestic fires or incinerators
 - Never flush them down the toilet



1




3




4



8

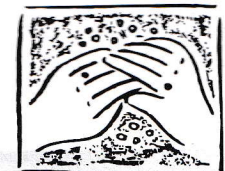

1

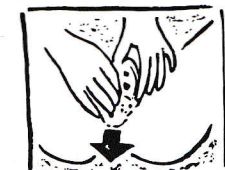

1



5



9

• Female

Before each catheterisation, make sure you have all you need together e.g. a new catheter, lubricating gel (if used), cleansing materials.

1. Wash your hands thoroughly and cleanse the genital area (as you have been shown by your Health Care Professional) and always from front to back.
2. Position yourself as you have been shown.
3. Remove the catheter from the packaging. You may wish to either insert lubricant into your urethra or apply the lubricant to the catheter, prior to insertion.
4. It may be beneficial to use a mirror with light to identify your urethral opening. (For product information please contact our customer services on 01494 532761).
5. Open the labia with two fingers. Using the other hand, gently insert the catheter into the urethral opening.
6. Insert the catheter until urine begins to flow. You may feel more resistance as the catheter enters the bladder. Do not worry, relax and push gently. Once urine flows, maintain the position of the catheter and try to avoid further insertion.
7. When the urine stops flowing, gently push the catheter a little further into the bladder and wait to see if more urine flows.
8. When you are happy that your bladder is empty, slowly withdraw the catheter. If more urine flows, stop and wait for it to cease, and then continue to withdraw.
9. Wash your hands thoroughly and discard the catheter in line with your local waste recommendations. Most Councils recommend first rinsing the used catheter and either seal in a plastic bag or wrap in newspaper before putting in the normal household waste.
 - Never burn any used catheters on domestic fires or incinerators
 - Never flush them down the toilet

Notes

Looking after yourself

Your Doctor or Nurse should be told if you notice any change in the smell, colour or clarity of your urine or if blood or particles are seen.

Delivery

Catheters are supplied individually packaged in a sterile pouch. The sterility of the products is only guaranteed as long as the packaging unit is unopened and undamaged, and no longer than indicated on the label.

Storage Instruction

The products shall be stored in their original packaging. Keep away from sunlight and keep dry. Do not use if the product sterilisation barrier or its packaging is compromised.



Warnings:

The products are for single use only.



Single use: Do not reuse, reprocess or re-sterilize. Reuse of device creates a potential risk of serious injury and/or infection which may lead to death. Reprocessing of medical devices intended for single use only may result in degraded performance or a loss of functionality.



REF 5143 and 5105

These catheters contain natural rubber latex (see product label), which may cause allergic reactions. Under no circumstances should these products be used in patients with a latex allergy.

Sterilization method: see product label



Explanation of important symbols and markings on the product label.

REORDER	Complete product order number, consisting of the order number "REF" and the product size "SIZE"
REF	Catalogue number
SIZE	Product size
Ch.	1 Ch. = 1/3mm
O.D.	Outer diameter
Length	Catheter length
Qty	Quantity, Number of items
LOT	Batch code
	Date of manufacture
	Use by
STERILE EO	Sterilized using ethylene oxide
STERILE R	Sterilized using irradiation
	Manufacturer
EC REP	EU Authorized Representative
	Caution, Attention, Important, Note
	Consult instructions for use
	Do not reuse
	Do not resterilize
	Not made with natural rubber latex
	This product contains natural rubber latex
	Do not use if package is damaged
	Keep dry
	Keep away from sunlight



Teleflex Medical Sdn. Bhd.
Lot PT 2577, Jalan Perusahaan 4,
34600 Kamunting, Perak, Malaysia
Made in Malaysia



Teleflex Medical
IDA Business & Technology Park,
Dublin Road, Athlone,
Co. Westmeath, Ireland

For further information please contact us on:

TELEFLEX MEDICAL EUROPE Ltd.
IDA Business & Technology Park
Dublin Road, Athlone, Co. Westmeath, Ireland
Phone +353 (0) 906 460 800
Fax +353 (0) 143 707 73

www.teleflex.com

