

This guidance outlines the correct use of Curion® CuriFlush for healthcare professionals, carers, or patients who have received appropriate training.

Preparation

1. Place a clean, absorbent towel on the patient's bed to manage any potential fluid spills.
2. Perform hand hygiene as per clinical best practices.
3. Maintain sterility – The Curion® CuriFlush sachet is supplied sterile and should remain in its protective wrapping until you are ready to administer.
4. Ensure correct temperature – The solution should be at body temperature (approx. 37°C). If required, immerse the sealed sachet in lukewarm water to achieve this.



Setup

1. Open the outer sterile packaging and remove the sachet.
2. **Close the clamp by securing it firmly to prevent any leakage. See Fig.1**
3. Snap off the narrow end of the blue cap and dispose. See Fig.2
4. Remove the blue cap from the connector by twisting it off, ensuring the clear sterile connector is not touched.



Fig.1

Administration

1. Insert the clear sterile connector into the catheter, applying a slight twist to facilitate secure insertion.
2. Open the clamp to initiate fluid flow, gravity-fed. See Fig.3
3. Allow fluid to infuse until no further flow is observed.
Do not apply pressure or manually squeeze the sachet.



Fig.2

Drainage & Disposal

1. **Facilitate drainage by positioning the bag downwards, enabling used solution to flow back into the CuriFlush sachet.**
2. Once drainage is complete, securely close the clamp and disconnect the Curion® CuriFlush from the catheter. See Fig.1
3. If required, attach an appropriate drainage bag or catheter valve.
4. Dispose as per infection control and waste management guidelines.
5. Perform hand hygiene as per clinical best practices.



Fig.3

