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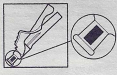
English

Español

1) Push firmly when attaching the needle to the syringe. Pull back on the safety cover. Grasp the syringe with one hand and with the other hand pull the needle shield straight off.

Bevel up = Safety Cover Up

Draw up medication and administer medication in accordance with established protocol.

2) Activate safety mechanism immediately after injection. Center your thumb or forefinger on the textured finger pad and push the safety cover forward over the needle until you hear or feel it lock. Visually confirm that the needle is covered. If unable to activate, discard immediately in an approved sharps container.

3) Use one handed technique and activate away from self and others. For greatest safety, ONLY use the wide textured finger pad area to activate the safety cover.

Activation of the protective mechanism may cause minimal splatter of any fluid that is remaining on the needle after injection.

Discard after single use in an approved sharps container in accordance with applicable regulations and institutional policy.

Non-pyrogenic. Do not use if individual packaging is damaged.

This product does NOT contain natural rubber latex. Do Not Reuse

Manufacturer

Authorized Representative in the European Community

CAUTION

Where local and/or institutional procedures permit/require transportation of the filled syringe, use a passive recapping technique to cover the needle before transporting to the point of administration.

Re-use may lead to infection or other illness/injury.

USA only: OSHA standards require that such recapping must be accomplished using a one handed technique. DO NOT hold the needle shield during the recapping process.

USA only: Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

To help avoid HIV (AIDS), HBV (Hepatitis) and other infectious diseases due to accidental needlesticks, activate the protective mechanism immediately after use.

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